

Pro-Matrix Curve

A fifty year old lady presented with a disto-occlusal cavity in her upper left first premolar (1).



1

After a local anaesthetic was administered a rubber dam clamp was applied to the upper left second premolar and the dam applied in the usual fashion .

The cavity was subsequently prepared and a large size Pro-Matrix Curved band was applied to the tooth (2). The Pro-Matrix Curved band was chosen as it provides a fabulous contact point to the adjacent tooth (3), whilst at the same allowing easier application than any of the currently available sectional matrix systems.



2

A wedge was applied between the upper left premolars and the restoration was then acid etched, bonded and restored with a nano hybrid filled composite.



3

The speed of application of the matrix together with versatility of using a matrix band system, compared to sectional matrices can not be underestimated in allowing larger restorations to be completed. There is no need to purchase expensive clamps or application forceps either! LIFE JUST GOT EASIER!!!

